

Sterisets

Medical Products

Preferred Medical Sets & Procedure Packs





Sterisets International BV was established in 1996, as a result of a spinoff from a medical distributing company in The Netherlands. Instead of only buying and selling, the goal was to start our own manufacturing facility in the medical business. We have been growing our activities worldwide, with currently 5 facilities in The Netherlands and in Portugal, which together form the STI group.



Sterisets

Oss, The Netherlands

This is the headquarter of the Sterisets International Group, with the management, international sales division and logistics department.



Steripack

Penafiel, Portugal - Logistics & Offices

With this new facility, opened in 2020 across the road from our production facility, we have expanded our warehouse capacity with 2.600 m², and 700 m² office space.



Steripack

Penafiel, Portugal - Production & Quality

Our 3.500m² facility, gives space to our cleanroom, injection molding production, warehouse and laboratory. In our 1.000 m², ISO 7 cleanroom, thousands of medical kits and trays are produced daily, under close supervision of our highly qualified quality department. The Prefilled syringes are made in a Cleanroom ISO-5, within the same facility.



Steripack

Guilhufe, Portugal - Non-sterile Production and Warehouse

At our first Portuguese facility, the production for the lamination and folding of our drapes and draw sheets and storage of our non-sterile products takes place.



Sterifluids

Cascais, Portugal - Medical fluids

The newest member of our family... Here we produce medical fluids for Dialysis, Disinfectant, Saline (NaCl) and other fluids. In this factory we also produce scrub brushes, nose sprays and pre-filled syringes.

Together, all of us at The Sterisets International Group (STI) are dedicated to provide you with a wide range of medical products to increase efficiency in the medical field.

Sterisets

Medical Products

Medical Sets & Procedure Packs

Sterisets® Medical sets & procedure packs, increase the efficiency of hospitals and reduce the cost of patient care. Prices are kept low by Sterisets own large scale, highly rationalized production methods.

Advantages:

- Decrease in contagion hazards.
- Easy transport and handling.
- Maximum hygiene.
- Savings on washing, cleaning and sterilization.
- Create less waste

Cleanroom - ISO 7

In our 1.000 m², ISO 7 cleanroom, thousands of medical sets & procedure packs are produced daily. All our sets & procedure packs are under close supervision of our highly qualified quality department.

Quality management system

Our Quality management system, is applicable to the design, development, manufacturing and distribution of single use medical devices. Also applicable to custom procedure packs, both for catheterization, disinfection, woundcare, dialysis and general surgical procedures.

Customized Sets

In this brochure we present to you our most common sets used. You can still customize your own sets using our component catalogue. The component catalog can be found on our website.



Always want to have access to the latest version of our brochures? All brochures and leaflets are available digitally on our website www.sterisets.com.


In case you prefer a physical copy of our artwork, Sterisets International Group uses Green Label paper, made out of 100% recycled paper with FSC quality mark. In addition, the paper is produced without usage of any type of chlorine (ECF).

Anesthesia Sets

Ref. 10601016



Label :

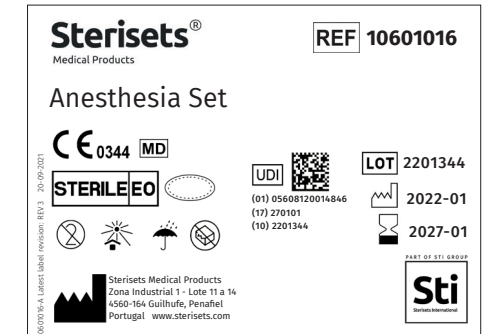
[illegible]

Bladder Catheterization Sets

Ref. 10605120



Label :

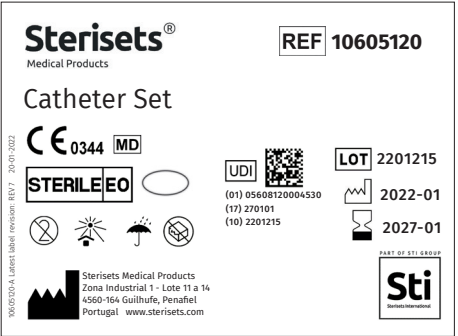


An impression of our Catheter sets																																													
	5844INT	5840INT	5813INT	56211	56203	56206	561420	561409	561406	217014INT	217011INT	219003	213100INT	213006INT	213003	1104	1112	1150	1148	1145	1149	1115	655132	655133	655133FC	655214	655215	56009PC5	56005	56005PC5	56405PC5	56402PC5	55110	14318	14318BSNS	14313BSNS	215021S	22021	1003	1007	1006	1015			
10605002 Catheterization set	1			4									1						1												2									1					
10605040 Catheterization set			1								1	1									1					1								9								1			
10605054 Catheterization set		1				6				1		1								1			2						2														1		
10605066 Catheterization set		1					5				1			1				1									2	2					1				1	1			1				
10605078 Catheterization set								5			1			1			1										1									1	2								
10605080 Catheterization set	1								5	1		1					1															5						1						1	
10605082 Catheterization set		1			5																	1		2	2						2				1	1		1							
10605120 Catheterization set		1		5																		1	2							1															

Dialysis Sets

Ref. 10606914



Label :

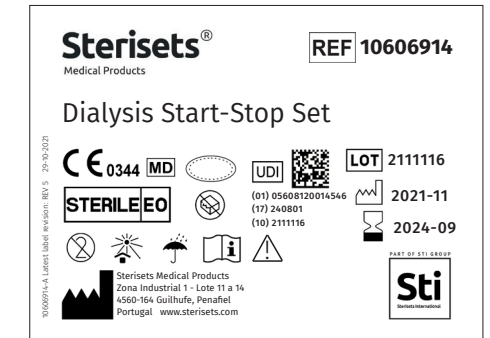

An impression of our Dialysis sets	Tray PP 180x95 x25mm 2 comp. transparent	Tray PP 247x138 x25mm 1 comp. blue	Absorbent pad nw/ pulp/ nw 10x10cm	Bib dental 38x30cm folded	Bandage cotton 25mx3cm	Cotton ball 0,6 gram	Cotton tip applicator wood small tip 15x0,2cm	Drape PE/ Tissue 38x43cm green	Drape PE/ Tissue 40x45cm blue	Drape PE/ Tissue 50x42cm green	Drape PE/ Tissue 50x60cm green	Drape PE/ Tissue 55x60cm blue	Drape 2 ply PE 33 gr/m2 NW 30 gr/ m2 50x70cm	Gallipot PP 120 ml Ø84mm height 30mm transp. with grad.	Needle Hypodermic 18G x 1,5" (1,2x40mm) TW, Pink	Needle Hypodermic 18G x 2" (1,2x50mm) TW, Pink	Strip Silk 12mm x100mm 3 strips per sheet	Swab gauze 8 ply 17 thr. 5x5cm counted per 5	Swab gauze 8 ply 17 thr. 5x5cm with Y-split	Swab gauze 8 ply 17 thr. 7,5x7,5 cm counted per 5	Swab NW 4 ply 30 gr 10x10cm counted per 5	Swab NW 4 ply 30 gr 5x5cm counted per 5	Swab NW 4 ply 30 gr 7,5x7,5 cm counted per 5	Syringe 3P Luer Lock 10ml, grad. 0,2ml, bulk	Syringe NaCl 0,9% Luer Lock 10ml, with label	Syringe 3P Luer Lock 5ml, grad. 0,2ml, bulk	Syringe 3P Luer Lock 3ml, grad. 0,1ml, bulk	Closure cap Luer Lock male/fe- male red	Closure cap Luer male white	Wrapping field PE/ Tissue 50x50cm green	Wrapping field PE/ Tissue 70x60cm green	Wrapping field PE/ Tissue 38x43cm blue		
																																		
Item / Description	213004INT	213021	56380	1091-1	56770	561406	56602	1101	1102	1103	1107	1104	3211	217014INT	304644	304636	4357	56001	56004	56005	56405	56400	56402	55010	14363	55005	55003	1810	1811	1003	1009	1002		
10606022 / Dialysis set						5			1									5																
10606024 / Dialysis set			3							2								4	2															
10606034 / Dialysis set			2						1											5														
10606077 / Dialysis set	1								2								2					15				2				1				
10606078 / Dialysis set		1											1	1			10														1			
10606087 / Dialysis set	1				0,017 mtr					1						2					7	6		2		2	2			1				
10606089 / Dialysis set	1				0,017 mtr		4	1													10	3			2	2				1				
10606912 / Dialysis fistula set	1			1					1											10				2									1	
10606914 / Dialysis start-stop set	1										1				2						6				2		2	2	2	2	1			
10606917 / Dialysis start-stop set	1											1									10				2		2			1				


Dressing Sets

Dressing Sets

Ref. 610159



Label :

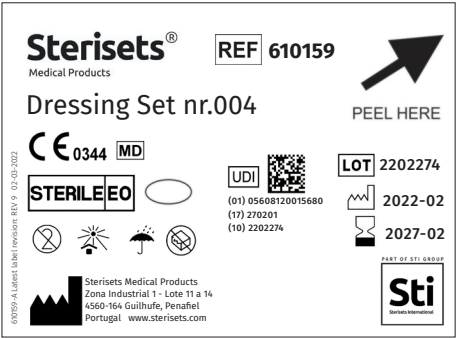


An impression of our Dressing sets with Hard blister	Ball gauze 17 thr. 20x20cm Ø30mm	Ball NW 13x15cm Ø35mm 30g/m2 with ring	Swab gauze 8 ply 17 thr. 5x5cm counted per 5	Swab gauze 8 ply 17 thr. 7,5x7,5 cm counted per 5	Swab NW 4 ply 30gr 5x5cm counted per 5	Swab NW 4 ply 30gr 7,5x7,5cm counted per 5	Forceps Burger plastic blue 12,6cm	Forceps Burger plastic green 12,6cm	Drape 2 ply PE 33gr/m2 NW 30gr/m2 40x45cm	Drape PE/Tissue 38x43cm blue	Drape PE/Tissue 40x45cm blue	Drape PE/Tissue 55x60cm blue
Item / Description	56211	56502	56001PC5	56005PC5	56400PC5	56402PC5	5844INT	5840INT	3215	1100	1102	1104
610019 / Dressing set nr. 001	5											
610059 / Dressing set nr. 002	3							1				
610159 / Dressing set nr. 004	5							1		1		
610209 / Dressing set nr. 005			5					1		1		
610259 / Dressing set nr. 006	4			1				1		1		
610409 / Dressing set nr. 009	5						1	1			1	
610459 / Dressing set nr. 010	5			5			1	1			1	
610509 / Dressing set nr. 014	5			1				1				2
610626 / Dressing set nr. 022		5				5	1	1			1	
610628 / Dressing set nr. 023					20						1	
610665 / Dressing set nr. 073		5				5	1	1	1			

Suture Sets

Ref. 10608046



Label :

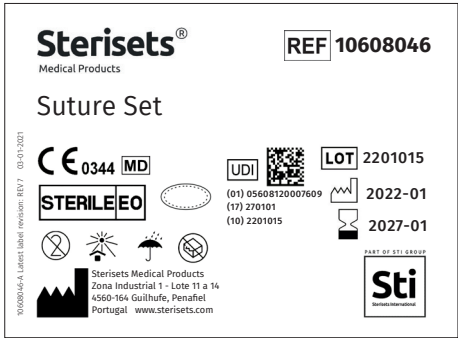


An impression of our Suture sets																																					
Item / Description	56201	56501	217014INT	219023INT	217018INT	213004INT	213015INT	570948S	3236	1115	1141	5875	5840INT	570037	570034	570038	570002	570033	570232	570743	570021	570124	570018	570059	570011	570012	570056	570016	56003	56402	56011	56005PC5	56405PC5	1003	1007	1006	1015
10608032 Suture set		10		1	1										1													1								1	
10608036 Suture set			1				1		1			1				1	1					1				1					10				1		
10608043 Suture set						1								1						1							1							1			
10608044 Suture Removal set						1								1										1						2					1		
10608045 Suture set																1	1					1	1												1		
10608046 Suture set			1							1	1		1																			10					1
10608070 Suture set						1									1				1						1										1		
10608088 Suture set	5					1									1					1				1									3	1			
10608094 Suture Removal set								1										1											3								

Eye Sets

Ref. 10617008



Label :

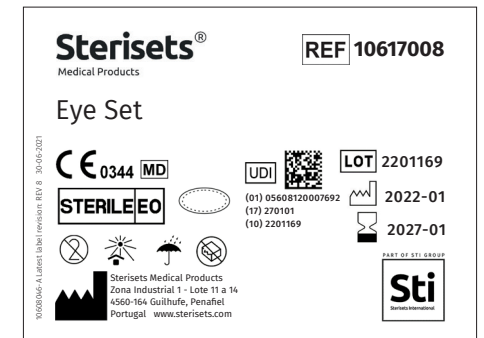[illegible]

Urine Sets

Ref. 310019



Sterisets® Medical Products **Uricol**
 The Newcastle Urine Collector

How to use the Uricol collection pack to collect a urine sample.


- 1 Ensure the patient's genital area and bottom is washed using soap and water, drying from front to back. Do not apply cream or talcum powder.
- 2 Ensure the Uricol pad is accurately positioned over the area where the patient passes urine. For a baby it may be helpful to attach the pad by fitting a cotton feel disposable nappy inside out: This is to ensure the nappy does not absorb any urine.
- 3 Check at 10-minute intervals to see if the pad is wet. Repeat this process every 10 minutes - If the pad is not wet within 1/2 hour YOU MUST apply a new pad and start again.
- 4 As soon as the pad is wet, remove it from the patient. If the pad is soiled, you will need to start the process again (from step 1).
- 5 Lay the pad on a flat surface, wet side up.

Continue on reverse

Uricol
 Urine Collector

STERISSETS INTERNATIONAL, S.A.
 Zona Industrial 1 - Lote 11 a 14
 4560-164 Gullufre, Penafiel
 Portugal www.sterisets.com

Sti
 Steris International

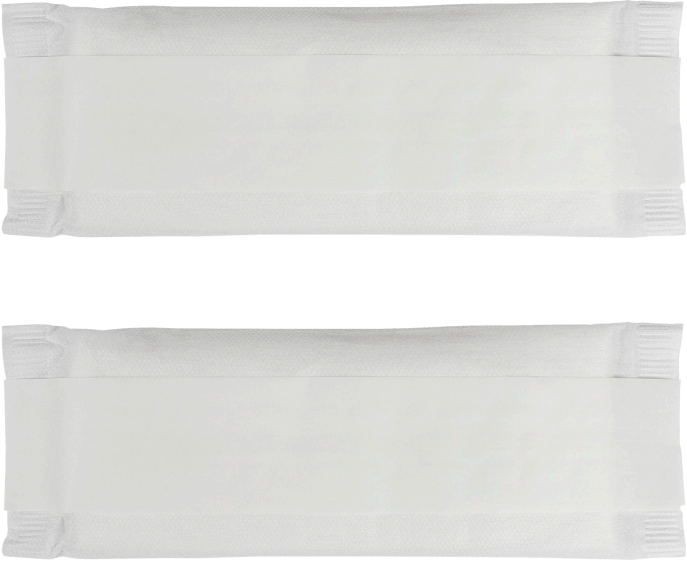

Label :

Sterisets® Medical Products **REF 310019**
 Urine Set

CE 0344 MD
 UDI
 LOT 2203281
 2022-03
 2027-03

(01) 05608120001904
 (17) 270301
 (10) 2203281

Sterisets Medical Products
 Zona Industrial 1 - Lote 11 a 14
 4560-164 Gullufre, Penafiel
 Portugal www.sterisets.com

Sti
 Steris International


An impression of our Urine sets	Bowl PP 250ml Ø101mm height 54mm transp. w. grad	Kidney bowl PP 300ml 214x120x43mm transp. with grad.	Kidney bowl PP 800ml , 264x150x56mm transp. with grad.	Swab gauze 8 ply 17 thr. 5x5cm	Absorbent under pad NW/PULP/PE film 60x40cm	Cotton ball 0,6 gram	Cotton ball 3 gram	Container PS 30 ml Ø 22 mm height 90 mm w. cap mounted	Syringe 3P Luer centric 5ml, grad. 0,2ml, bulk	Syringe 3P catheter tip 50-60 ml grad. 2,0 ml bulk	Maternity pad NW/Viscose PE film w. strip 23x7,5 cm	Data sheet Uricol	Wrapping field PE/Tissue 70x60cm green
Item / Description	217011INT	219024INT	219026INT	56001	56970	561406	561430	218030	55105	55350	1700		1009
310019 Urine set								1	1		2	1	
10605081 Urine set		1	1		1					1			1
10605111 Urine set	1					4							
10605113 Urine set	1			1									
10605114 Urine set - Examination	1	1					3						

Pain Sets

Ref. 10699012



Label :

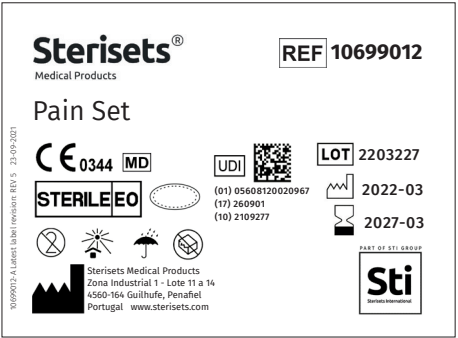

An impression of our Pain sets	Ball gauze 17 thr. 20x35 cm Ø40mm with elastic ring	Swab gauze 8 ply 17 thr. 5x5cm counted per 5	Swab gauze 8 ply 17 thr. 7,5x7,5cm counted per 5	Bowl PP 250ml Ø101mm height 54mm blue w. grad.	Gallipot PP 60 ml Ø63mm height 30mm transp. w. grad.	Tray PP 193x130 x36mm 3 comp. transp.	Tray PP 196x133x 36mm 1 comp. transp.	Tray PP for instrument 270x135 x25mm 1 comp blue	Drape 2 ply PE 33 gr/m2 NW 30gr/m2 75x75cm lateral adhesive	Drape 2 ply PE 33 gr/m2 NW 30 gr/m2 75x90cm lateral adhesive	Drape 2 ply PE 33gr/m2 NW 30 gr/m2 75x90 cm	Foam (blue) stick (white) 16cm	Needle Hypodermic 18Gx1,5" (1,2x40mm) TW Pink	Needle Hypodermic 21Gx2" (0,8x50mm) RW Green	Needle Hypodermic 25Gx1" (0,5x25mm) RW Orange	Needle Blunt Fill 18Gx1,5" (1,2x40mm) 5µ filter Purple	Syringe 3P Luer centric 10ml, grad. 0,2ml bulk	Syringe 3P Luer Lock 10ml, grad. 0,2ml, bulk	Syringe 3P Luer centric 5ml, grad. 0,2ml, bulk	Syringe 3P Luer Lock 5ml, grad. 0,2ml, bulk	Syringe 3P Luer centric 2 ml, grad. 0,1 ml, bulk	Syringe 3P Luer Lock 3 ml, grad. 0,1 ml, bulk	Plaster dressing absorbent 7x9cm adhesive	Close seal	Towel airlaid 21x25cm 80 gr/m2	Probe double ended 3,5mm x25 cm satin	Wrapping field 2 ply PE 33gr/m2 NW 30gr/m2 70x90cm blue	Wrapping field 2 ply PE 33gr/m2 NW 30gr/m2 75x75cm blue	Wrapping field PE/Tissue 70x90cm blue	Wrapping field 2 ply PE 33gr/m2 NW 30gr/m2 75x110cm blue	
																															
Item / Description	56205	56001PC5	56005PC5	217018INT	217006INT	213016INT	213015INT	213013	3225	3224	3213	5880	304644	304630	300403	305236	55110	55010	55105	55005	55102	55003	4303	22021	71600	590060	3203	3202	1015	3205	
10616012 Pain set	4								1											1		1						1			
10616043 Pain set			5						1				1				1						1							1	
10699012 Pain set		5		1	1		1			1		1	1	1	1	1		1	1		1				1						1
10699019 Pain set			2					1			1		1							1		1		1			1	1			
10699027 Pain set			5		1	1				1		2	1	1	1					1		1					1				
10699037 Pain set		5						1		1			1					1		1							1	1			

Sterisets

Medical Products

Sterisets International BV

Ketelmeer 3 | 5347 JX Oss | The Netherlands

P +31 (0)412 667 755 | E info@sterisets.com

www.sterisets.com

PART OF STI GROUP

Sti
Sterisets International